CE FDA

PrimeCast™

Orthopaedic Synthetic Casting Tape

- ◆ **Excellent Moulding**
Superior multi-directional stretch gives excellent moulding on curved area.
- ◆ **Smooth Finish**
High quality knitted substrate and special resin formula provides smooth surface and clean and strong end lamination.
- ◆ **Cost Effective**
Savings from less number of rolls needed and less re-casting compared to POP bandage.
- ◆ **Choice of Fiberglass and non-Fiberglass substrate**
For those who prefer benefits of non-fiberglass casting, high quality polyester-fabric casting tape is available.
- ◆ **Other advantages of Synthetic Casting Tape**
Lightweight, Strength and Durability, Air-permeability and X-ray transparency.

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112SW18E41

PrimeCast™

Orthopaedic Synthetic Casting Tape

PrimeCast™ is a synthetic casting tape impregnated with polyurethane resin which is activated by water before use. Quality of PrimeCast™ is well proven through years of successful sales worldwide and it is certainly one of the top quality products currently available in the market. Allows controlled motion functional loading accelerates healing

PrimeCast™ Soft is a semi-rigid (flexible) casting tape which allows controlled movement to accelerate healing of the wound for some applications. It may be used combined with standard rigid casting tape to provide movement only in a specific area.

How To Use

1. Apply a stockinette and a cast padding over the desired area to be cast.

2. Wear protective gloves.
3. Immerse PrimeCast™ in water for 1-2 seconds.


4. Squeeze the tape gently to remove excess water and wrap the desired area with the tape. Do not apply tightly.


Moulding and forming should be done at this time. Setting time is approximately 3-5 minutes. Functional strength will be achieved in 20 minutes.

Available Colors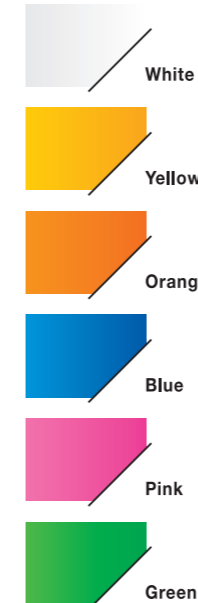

Ordering Information

Ref. No.	Description	Size	Rolls/Inner Box	Inner Boxes/Outer Carton
F 2	PrimeCast™ Fiberglass 2in	5.0cm x 3.6m (2in x 4yd)	10Rolls/Box	9Boxes/CTN
F 3	PrimeCast™ Fiberglass 3in	7.6cm x 3.6m (3in x 4yd)	10Rolls/Box	9Boxes/CTN
F 4	PrimeCast™ Fiberglass 4in	10.2cm x 3.6m (4in x 4yd)	10Rolls/Box	9Boxes/CTN
F 5	PrimeCast™ Fiberglass 5in	12.7cm x 3.6m (5in x 4yd)	10Rolls/Box	9Boxes/CTN
P 2	PrimeCast™ Polyester 2in	5.0cm x 3.6m (2in x 4yd)	10Rolls/Box	9Boxes/CTN
P 3	PrimeCast™ Polyester 3in	7.6cm x 3.6m (3in x 4yd)	10Rolls/Box	9Boxes/CTN
P 4	PrimeCast™ Polyester 4in	10.2cm x 3.6m (4in x 4yd)	10Rolls/Box	9Boxes/CTN
P 5	PrimeCast™ Polyester 5in	12.7cm x 3.6m (5in x 4yd)	10Rolls/Box	9Boxes/CTN
S 2	PrimeCast™ Soft (Flexible) 2in	5.0cm x 3.6m (2in x 4yd)	10Rolls/Box	9Boxes/CTN
S 3	PrimeCast™ Soft (Flexible) 3in	7.6cm x 3.6m (3in x 4yd)	10Rolls/Box	9Boxes/CTN
S 4	PrimeCast™ Soft (Flexible) 4in	10.2cm x 3.6m (4in x 4yd)	10Rolls/Box	9Boxes/CTN
S 5	PrimeCast™ Soft (Flexible) 5in	12.7cm x 3.6m (5in x 4yd)	10Rolls/Box	9Boxes/CTN

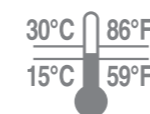STORAGE

Prime
MEDICAL INC

Prime Medical Inc. Korea
189 Gumdang-ri, Mado-myon, Hwasung-si,
Gyeonggi-do, Korea
Tel. +82-31-355-8374 / Fax. +82-31-355-8320
sales@primemedical-inc.com
www.primemedical-inc.com

Prime Medica Promotion Pvt. Ltd. India
107, Neelkanth Estate No. 1 Sativali,
Vasai (E), Dist: Thane-401 208 Maharashtra, India
Tel. +91-250-3291725 / 6526135
Fax. +91-250-2480551
india@primemedical-inc.com